


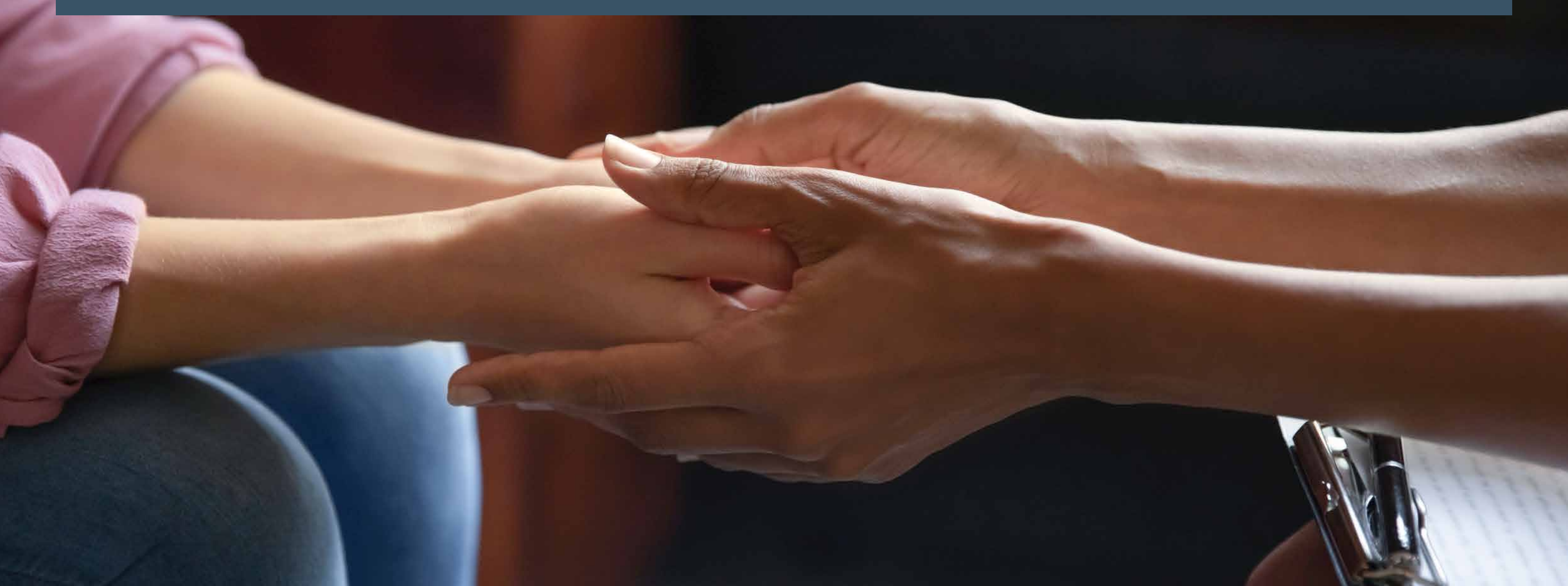

Beds for psychiatric care

Modern concepts to satisfy many requirements

Solutions for people who care



Beds designed for psychiatric care

Safety, a feeling of security and easy handling, even in difficult situations – these are the requirements for beds in psychiatric facilities. The beds must be designed to withstand greater physical demands than other medical products as well as provide protection and a comfortable, homelike feel, where this contributes perceptibly to the healing process. Stieglmeyer offers convincing solutions to meet these needs. Discover the tailor-made models we offer for:

- Geriatric psychiatry
- Child and adolescent psychiatry
- Psychosomatic medicine
- Special requirements (e.g. prison hospital, forensic medicine)

Special beds from Stieglmeyer

Throughout its 125-year company history, Stieglmeyer has been an important force in shaping the development of beds for hospitals and care facilities. Our beds set high benchmarks – whether for patient mobilisation, automated reprocessing or modern operation and controls. In the field of psychiatry, too, you can always rely completely on our robust and safe hygienic products to support your work.

Contents

Geriatric psychiatry	4	Psychosomatic medicine	20
• Puro – The hospital bed to meet exacting demands	6	• Libra – Specially designed to meet a wide range of individual needs	22
• Tereno – The care bed for floor-level resting and sleeping	8	• Carino – Ideas for comfort and a homelike feel	24
• Elvido rega – The mechanical care bed	10	• Performance characteristics of beds for psychosomatic medicine	25
• Elvido vervo – The versatile low-height bed	11		
• Classiko – The robust care bed	12	Restraint beds for security-sensitive areas/prison hospital	26
• Performance characteristics of gerontopsychiatric beds	13	• Deka – The stable bed for challenging everyday conditions	28
		• Performance characteristics of Vivendo pro	29
Child and adolescent psychiatry	14		
• Seta pro junior – The hospital bed for young people	16	Useful extras	
• Carino – The flexible bed for adolescent psychiatry	18	• Quado, Aparto bedside cabinet, furniture	30
• Performance characteristics of beds for child and adolescent psychiatry	19	The Stieglmeyer Group	31



Geriatric psychiatry

Meeting the needs of the elderly

People are living to an increasing age – which is a pleasing development. But with increasing life expectancy comes a growing risk of limitations and illnesses which can impair people’s physical and mental health.

Today, medical research on ageing is aware of the close relationship between mental and physical health. But this equation is all too often overlooked, particularly where older people are concerned. Ageing is not a condition that occurs suddenly but is a process that affects a person’s whole life. Stiegmeyer places a particular focus on residents and patients in old age – with products that are adapted to meet the changing needs, limitations and life perspectives of ageing persons and to take into account their restricted mobility and cognitive performance in a gerontopsychiatric setting.

The Stiegmeyer Group can look back on 125 years of experience as a manufacturer and supplier to the healthcare sector. With innovative beds and furniture crafted from high-quality materials, we are a comprehensive solution provider for clinical, inpatient and home settings. A holistic focus on mental, physical and social aspects – that is what our portfolio of gerontopsychiatric products stands for.



Puro
The hospital bed to meet exacting demands

- Height adjustment range: 32 - 91 cm (depending on castors)
- Safe working load: 275 kg
- External dimensions approx. 103,5 x 218,5 cm



Tereno
The bed for floor-level resting and sleeping

- Height adjustment range: 15 – 80 cm
- Safe working load: 225 kg
- External dimensions (depending on wooden surround): approx. 104 x 211 cm



Elvido rega
The mechanical care bed

- Height adjustment range: 39 – 80 cm
- Safe working load: 225 kg
- External dimensions approx. 100,3 x 211 cm



Elvido vervo
The versatile electric care bed

- Height adjustment range: 25 – 80 cm
- Safe working load: 225 kg
- External dimensions (depending on wooden surround): approx. 104 x 211 cm



Classiko
The robust mechanical care bed

- Height adjustment range: 40 – 80 cm
- Safe working load: 225 kg
- External dimensions approx. 101 x 208 cm



Puro

The hospital bed to meet exacting demands

With its comfort, safety, easy handling and excellent hygiene properties – the Puro is more than just a bed. It also satisfies the most exacting requirements in psychiatric care. Its split Multiflex⁺ safety sides offer flexible protection while still leaving enough room for unobstructed entry and exit from the bed. The optional machine-washable version of the bed supports every facility in its fight against multi-resistant pathogens.

Special features

- Securing of headboard and footboard against inadvertent removal
- Handset can be locked away, and lockable compartment for the control panel and control box
- Mattress base with permanently screw-fixed elements
- An anaconda cable protects against strangulation
- Posey belt fixing points on all elements of the mattress base, including the seat section
- Low-niche design and concealed cable routing



- ▲ Standard sliding rails with cut-outs for belts make it possible to restrain patients if they present a danger to themselves or others.



- ▲ The lockable handset compartment can be used to store the LCD handset to prevent inappropriate use.



- ▲ The control box can be locked away to prevent inadvertent adjustments to the bed.



- ▲ Belt fixing points at seat section height.



- ▲ The anaconda cable is a mains cable that is protected by a fixed jacket to prevent it being misused by suicidal patients.



Supportive:

- Back-friendly working height of up to 80 cm
- Chassis without longitudinal struts allows easy floor cleaning and the use of patient lifts



Forward thinking:

- Flexible concept for side elements, from homelike to mobilisation-friendly to protective
- Headboards and footboards that can be removed without tools facilitate daily care



Tereno

The bed for floor-level resting and sleeping

The use of freedom-restricting measures runs contrary to the fundamental principle of care, which is to promote mobility. Enabling patients to take part in social life independently while maintaining a positive perception of their own body is only possible if their radius of action is restricted as little as possible. But, particularly in older age, this can increase the risk of falls. In geriatric psychiatry, in particular, the Tereno ultra-low bed solves this dilemma. It not only lets patients rest and sleep safely at a floor-level height of 15 cm but also allows them freedom of movement and promotes their individual capabilities.

Special features

- Avoids the use of deprivation of liberty safeguards (safety sides, restraints)
- Supports the patient's urge to move
- Reduces the potential risks of falls
- Correct and convenient working height for nursing staff thanks to height adjustment of up to 80 cm
- Respectful communication at eye level with the patient is promoted by individual height adjustment and a sitting position



Minimises the effects of falls:

- Ultra-low mattress base position, with a fall protection mat
- No danger of limbs getting caught under the bed
- Intermediate stops at an ergonomic height for getting out of bed, and before the bed moves to the ultra-low position



Prepared for digital systems:

- Out-of-Bed system and future connectivity functions can be used



Sophisticated safety:

- Automatic braking when the lowest mattress base position is reached
- Basic protection that inhibits falling out even when the safety sides are not raised





Elvido rega

The mechanical care bed

The Elvido rega care bed offers comfort without cables and uses practically no components that can be removed without the use of tools. This makes a significant contribution to the safety of patients who have serious psychiatric conditions or dementia.

Special features

- Height adjustment using an optional removable pump pedal
- Robust metal mattress base with welded-in mesh base
- Optional posey belt securing brackets on the rests and frame profiles of the mattress base
- Ideal for locations with no power supply



▲ If desired, the screw-fixed pump pedal can be secured by a magnet system to allow removal without tools.



Elvido vervo

The versatile low-height bed

Vario Safe


With its large adjustment range, the Elvido vervo low-height bed protects against fall injuries without the need for deprivation of liberty safeguards and thus promotes the well-being of residents. At its highest position, it also allows care staff to work in a back-friendly position. The optional economical 24-volt drive system ensures an enhanced degree of safety.

Special features

- Optional Vario Safe system for quick removal and replacement of safety sides, panels, headboards and footboards
- Optional securing of Vario Safe locks against tool-free removal of the elements
- Availability of an integrated Out-of-Bed system
- Individually selectable homelike wood decors and headboards/footboards



▲ The locks of the Vario Safe elements can be secured, if required, to prevent unintentional removal of the elements.

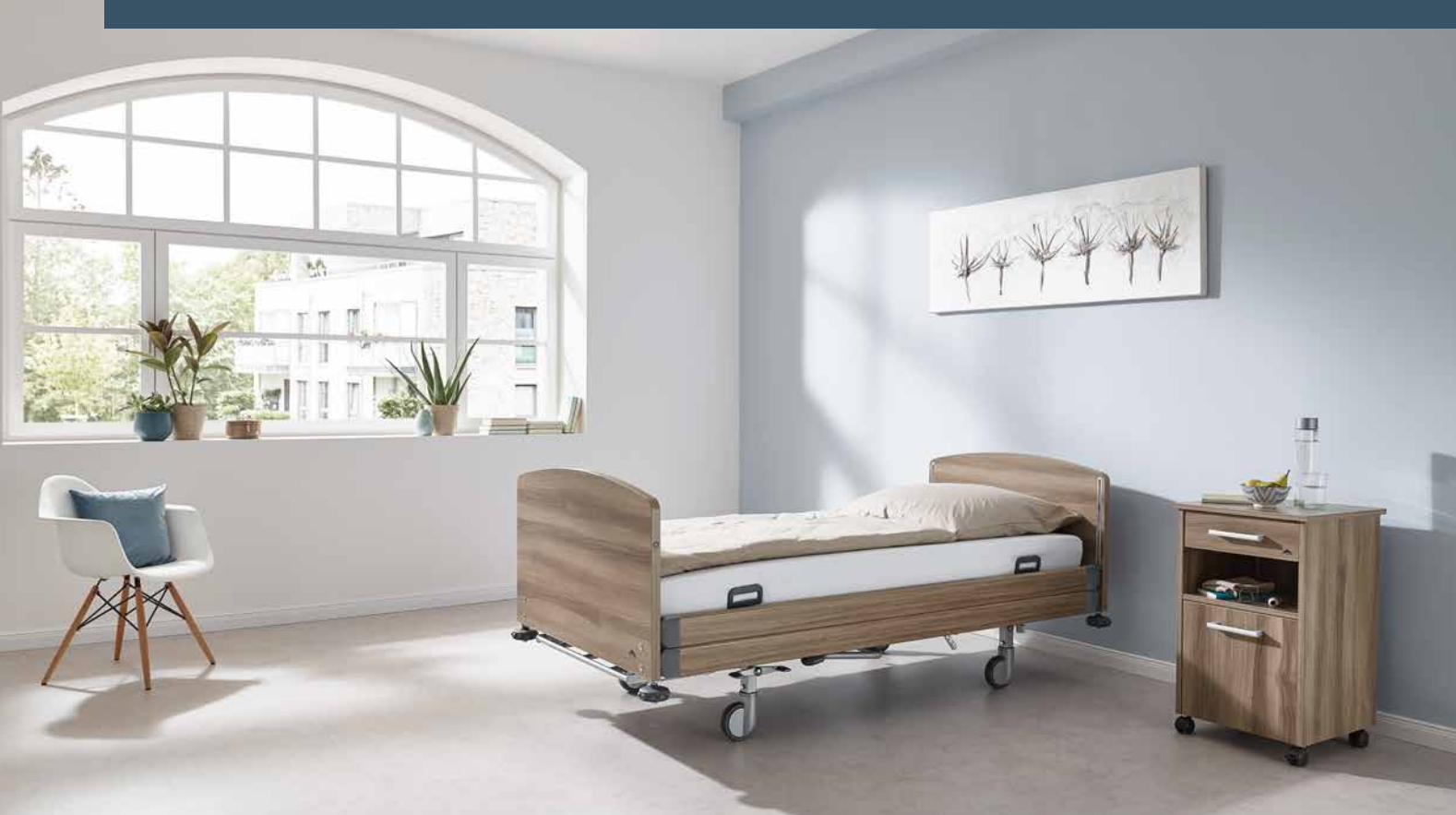

Classiko

The robust care bed



Comfort and safety for the resident, alleviation for care staff, and an individual design – the Classiko unites many strengths to give good value for money. By dispensing with cables and manually removable elements, this mechanical bed meets the safety requirements in psychiatric care particularly well. Care staff will find the mechanical Classiko easy to operate. Height adjustments using a pedal with the help of a hydraulic pump can be made just as easily as the power-assisted adjustments of the backrest and thigh rest.

Special features

- Hydraulic height adjustment for ergonomic working
- Full-length safety sides for maximum protection
- If necessary, removable pump pedal for height adjustment
- Optional securing brackets for posey belt systems on the mattress base
- 125 mm castors for easy manoeuvring even in constricted spaces
- Choice of attractive headboards/footboards and decors



▲ Pump pedal can be removed as required.



▲ Optional securing brackets on mattress base frame.




▲ Brake lever permanently secured with screws.

Performance characteristics of gerontopsychiatric beds

	Puro	Tereno	Elvido rega	Elvido vervo	Classiko
Wooden surround	–	✓	✓	✓	✓
Mattress base	Wire mesh mattress base, secured	Metal slat mattress base	Wire mesh base, fully welded in, 4 sections	Hygiene mattress base for belt systems	Wire mesh base, fully welded in, 4 sections
Lockable thigh rest and lower leg rest	–	–	0	–	0
Securing brackets, screw-fixed	–	–	0	–	0
Securing brackets on the seat section	0	–	0	–	0
Pump pedal secured, removable without use of tools	–	–	0	–	0
Brake pedal screw-fixed	–	–	0	–	0
Headboard and footboard					
Rondo, permanently screw-fixed, for MultiFlex* only	0	–	–	–	–
Stelo, permanently screw-fixed, for MultiFlex* only	0	–	–	–	–
Safety sides					
Full-length safety side (option of permanent fixing)	0	–	✓	✓	✓
Combinable safety side (option of permanent fixing)	–	–	0	0	0
Split safety side (option of permanent fixing)	–	–	–	0	–
Split safety side MultiFlex*	✓	–	–	0	–
Pivoting safety side, including horizontal telescope	–	✓	–	–	–
Vario Safe	–	0	–	0	–
Internal adapter sleeves for patient lifting poles					
Oval or round pole	✓	✓	0	0	0
Additional features					
Height adjustment	✓	✓	✓	✓	✓
Mattress base tilt	✓	✓	0	✓	0
With castors and central castor locking	✓	–	✓	0	✓
Lockable compartment in linen holder, for the locking box and handset	0	–	–	–	–
Lockable compartment for the handset	0	–	–	–	–
Restraining rail below backrest	0	–	–	–	–
Restraining rail at the head end and foot end	0	–	–	–	–
Mains cable as an "anaconda" cable	0	–	–	–	–
Bed extension	✓	✓	0	0	0
Linen holder	✓	–	0	0	0

✓ = Standard feature
0 = Available as an option
– = Not available

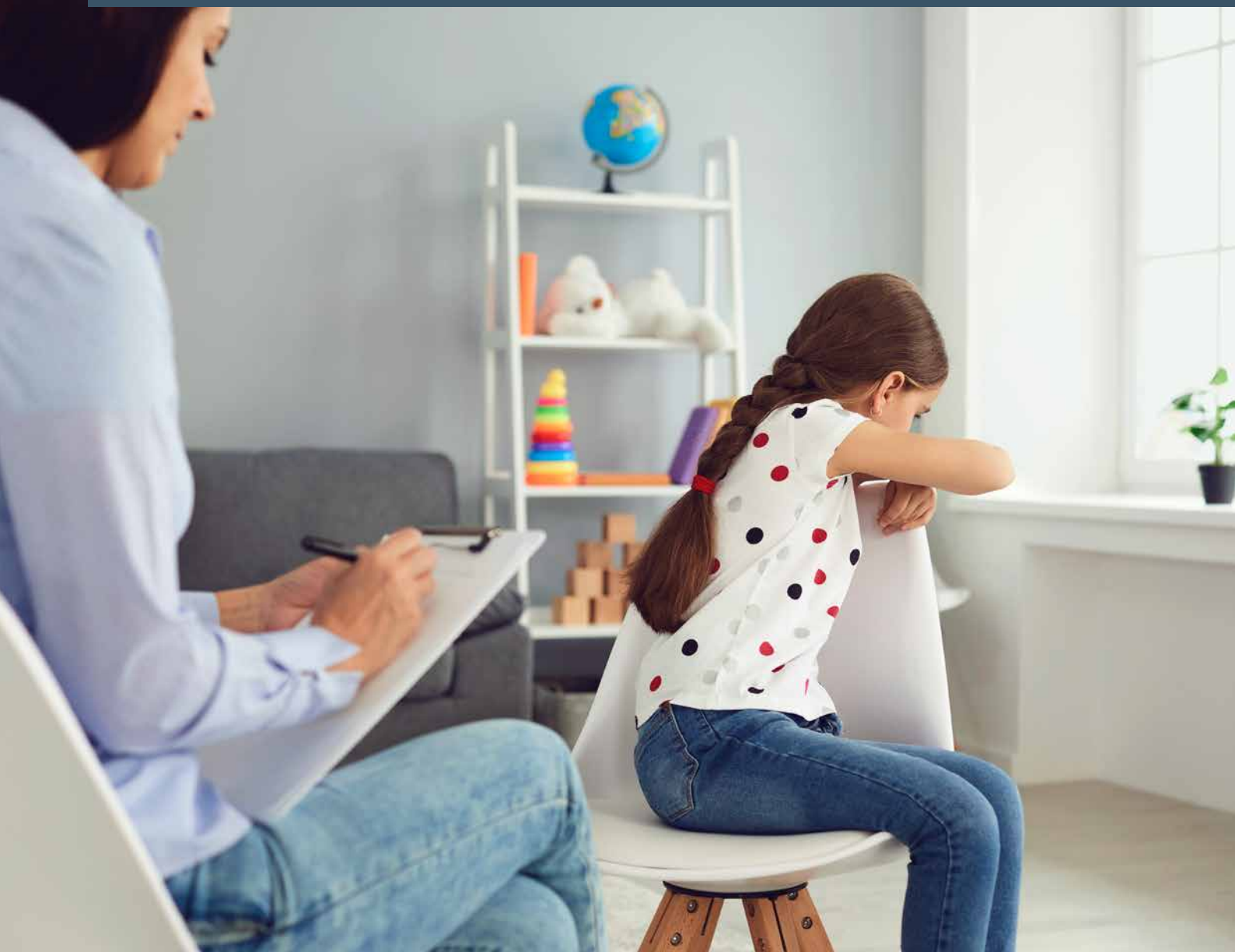

Child and adolescent psychiatry

Meeting needs when children and adolescents experience psychological disorders

Many countries are witnessing a continuing deterioration in the quality of life and mental health of children and adolescents. They observe increasing signs of depressive symptoms, psychosomatic complaints and mental health problems.

If an inpatient stay in a psychiatric facility is deemed necessary, the therapeutic support provided there can last a matter of days or weeks or up to a few months. The Stieglmeyer product portfolio for child and adolescent psychiatry is designed for many different uses ranging from clinical settings and rehabilitation facilities to special residential homes.

Our beds for use in child and adolescent psychiatry provide a safe and protected retreat and offer comfort and a wide-range of design options.



Seta pro junior
The hospital bed for young people

- Suitable for children with a height of up to 155 cm and a body mass index of less than 17
- Mattress base dimensions: 78 x 180 cm
- Height adjustment: 35 – 91 cm
- Safe working load: 175 kg
- External dimensions: approx. 93 x 193 cm



Carino
The homelike youth bed

- Mattress base dimensions: approx. 90 x 200 cm or 100 x 200 cm
- Mattress base height: approx. 40 cm
- Safe working load: 225 kg
- External dimensions: approx. 105 x 210 cm or 115 x 210 cm



Paediatric bed
standard
EN 50637-
compliant



▲ In a lockable compartment, the optional handset is protected from unintentional use.



▲ The linen holder can be slid into its storage position and locked in place. This ensures that the control box is safely stowed away from improper use.

Seta pro junior

The hospital bed for young people

Children and adolescents feel a particular need for warmth and security when they are in hospital – along with space to unfold and thrive. The Seta pro junior is a place to feel at home in and promotes speedy recovery. With its 78 x 180 cm mattress base, it provides ample room for children right up into their teens and is also easy to manoeuvre even in small rooms. The bed offers great comfort and convenience due to its fully motorised adjustments. Where patients are restricted in their mobility, the full-length safety side provided as standard protects them against injuries from falls. This can be operated effortlessly by nursing staff.


Special features

- EN 50637-compliant
- Full-length safety side integrated in headboard and footboard for maximum protection
- Secured mattress base elements
- Retaining element for handset and control box

High safety thanks to the EN 50637 standard

The Seta pro junior includes a number of intelligent details which implement the requirements of the new standard. Protective locking elements on the bed frame, for example, ensure that young patients are unable to slip through or become trapped anywhere.



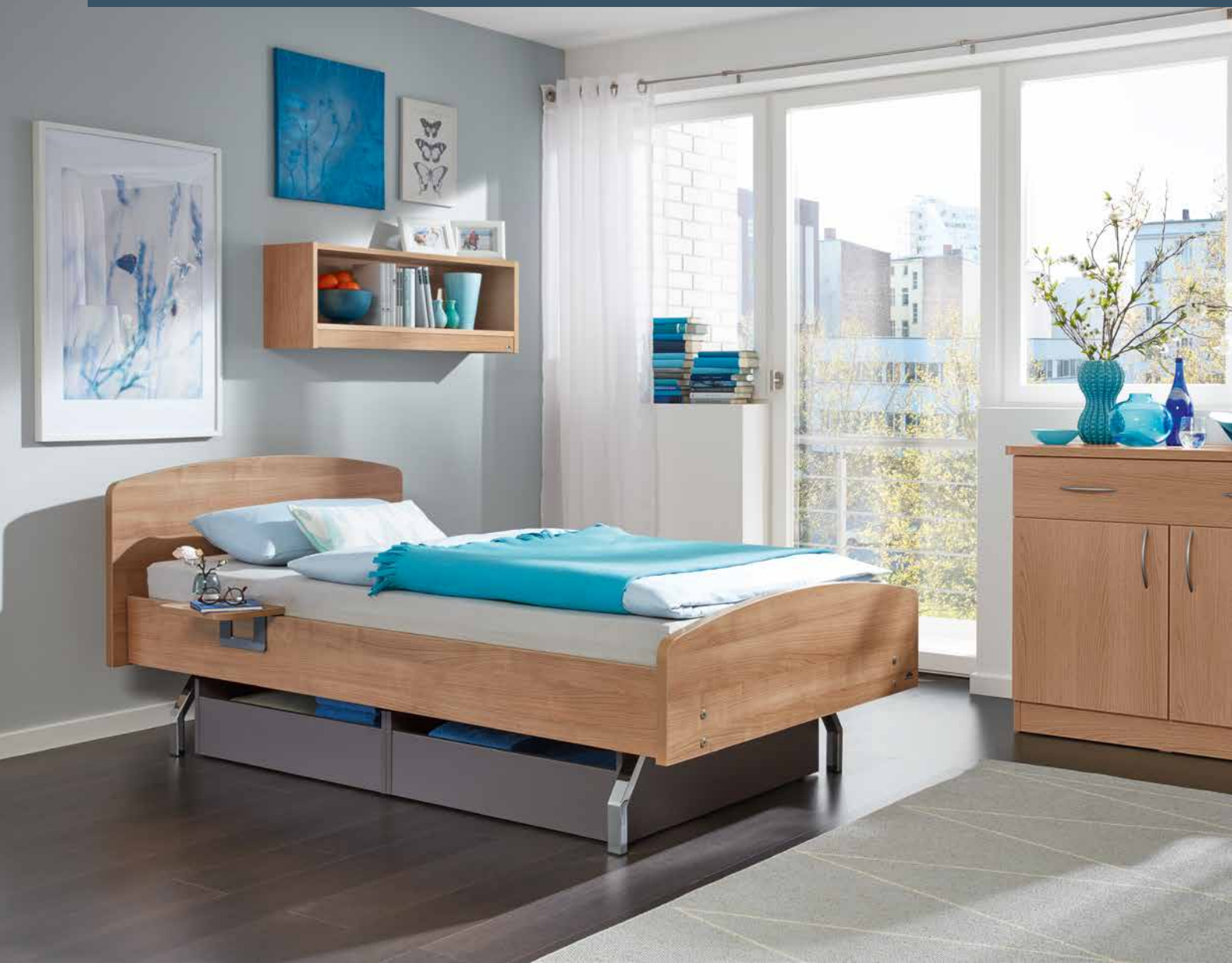

Carino

The flexible bed for adolescent psychiatry

The Carino is a bed with wide-ranging applications. It ensures quality of life in adolescent psychiatric facilities as well as in psychosomatic treatment centres, rehabilitation facilities and residential homes.

The Carino is particularly appreciated for its very good price-performance ratio and individual design possibilities. With its robust metal mattress base, the bed easily withstands numerous cycles of use. Mattress base widths of 90 and 100 cm are available and a bed extension can also be supplied for taller users.

- Special features**
- Manual adjustment of the two-section mattress base and backrest
 - Backrest adjustment to approx. 30°
 - Easy and hygienic reprocessing
 - Choice of wood decors
 - Chrome-plated feet
 - 2 adapter sleeves at the head end for patent lifting pole/reading lamp

▼ The metal slats of the mattress base are easy to clean.



Performance features for beds for child and adolescent psychiatry

	Seta pro junior	Carino
Wooden surround	–	Reinforced (screw-fixed Multiplex headboard/footboard and panels)
Safety sides		
Full-length	✓	–
Headboard and footboard design		
Base, not removable	✓	–
Mattress bases		
Metal slats, 2-section	–	✓
PP hygiene mattress base, 4-section	✓	–
PP hygiene mattress base for belt system, 4-section	0	–
Control elements		
Control box in the lockable linen holder	✓	–
1 standard handset, including 2 compartments	0	–
1 standard handset, illuminated, including 2 compartments	0	–
1 LCD handset, including 2 compartments	0	–
Additional features		
Linen holder, lockable, including control box	✓	–

✓ = Standard feature
 0 = Available as an option
 – = Not available



Psychosomatic medicine

Meeting needs when the mind responds with physical illness

Psychosomatic medicine is concerned with the interaction between biological, psychological and social factors. That's why Stiegemeyer places the focus on a homelike, comfortable and high-quality design when it comes to its products for use in this field.

Whether we're talking of a psychological crisis, burnout or mobbing – the human psyche often suffers in silence. Professional support and comfortable surroundings can help people to find their way back to a carefree life.



Libra

Specially designed to meet a wide range of individual needs

- Mattress base dimensions: approx. 90 x 200 cm or 100 x 200 cm
- Height adjustment range: 25 – 80 cm
- Safe working load: 225 kg
- External dimensions:
 - Combinable split safety side:
For mattress base width of 90 cm: approx. 100,3 x 209,4 cm
 - Full-length safety side (depending on the wooden surround)
For mattress base width of 100 cm: approx. 110,3 x 209,4 cm
 - Split safety side:
For mattress base width of 90 cm: approx. 104,0 x 209,4 cm
For mattress base width of 100 cm: approx. 114,0 x 209,4 cm



Carino

The attractive and flexible bed

- Mattress base dimensions: approx. 90 x 200 cm or 100 x 200 cm
- Mattress base height: approx. 40 cm
- Safe working load: 225 kg
- External dimensions: approx. 105 x 210 cm or 115 x 210 cm



Libra

Specially designed to meet a wide range of individual needs

The Libra is a bed that develops with you by keeping pace with your needs and taste. With its flexible Vario Safe system, the headboard and footboard, along with the side panels and safety sides, can be removed or replaced within seconds.

Would you like to change the design or enjoy more protection while you sleep? Then use the available options to tailor the bed to your requirements.

Special features

- Safety side elements and panels in different lengths
- Effortless control with intuitive handset
- Comfort mattress base for restful sleep
- Chair position for reading and watching television
- Attractive wood decors and fixed upholstery for every furnishing style



The Libra bed, a low-height bed with optional split safety sides, offers both a high level of safety and full freedom of movement. Its lowest position minimises the risk of fall injuries, and the raised split safety sides at the head end leave enough space to get in and out of bed. This is our way of following the Werdenfeller approach, which avoids the use of liberty deprivation measures (DoLS).

Special features

- Optional split safety sides for individual protection in any situation
- Tool-free removal of all safety sides with the Vario Safe or Easy Click system
- Homelike appearance thanks to a choice of headboards and footboards, wood decors and upholstery



Carino

Ideas for comfort and a homelike feel

Stable, durable, comfortable and extremely versatile – the Carino is a bed with many uses and design options. Its very good price-performance ratio ensures high quality of life in psychosomatic treatment centres.

The Carino has a sleek and elegant design and can be adapted to your own personal furnishing taste for the room with a wide range of available headboard and footboard forms and wood decors to choose from.

The textile Softcovers for the headboard, footboard and sides of the bed are an especially cosy design feature of the bed. Fabrics in attractive colours and patterns can be used to transform the Carino into the centrepiece of the room.

With an optional fixed board along the side of the bed, the Carino can also be used as a comfortable sofa during the day.

Special features

- Manual adjustment of the two-section mattress base and backrest
- Easy and hygienic reprocessing
- Backrest adjustment to approx. 30°
- Chrome-plated feet
- 2 adapter sleeves at the head end for patient lifting pole/reading lamp
- Space-saving side tray as practical accessory



Performance characteristics of beds for psychosomatic medicine

	Libra	Carino
Choice of wooden surrounds	✓	Reinforced (screw-fixed Multiplex headboard/footboard and panels)
Safety side systems		
Full-length safety sides (DSE) Easy Click	✓	–
Combined safety side (KSE) Easy Click	0	–
Split safety side with Vario Safe	0	–
No safety side	0	✓
Mattress bases		
Metal slat mattress base	✓	–
Metal slats, 2 sections	–	✓
Mattress base in four sections, electrically adjustable	✓	–
Mattress base tiltable to reverse-Trendelenburg position	✓	–
Mattress base tiltable to Trendelenburg position	0	–
Comfort mattress base inserts	0	–
Hygiene mattress base	0	–
Castors		
Double castors	Ø 50 mm	–
Castor braking, per axle	✓	–
Castor braking, central	0	–
Electronic components		
Handset (selective locking function)	✓	–
Additional handset for Trendelenburg position	0	–
Under bed light	0	–
Backlit handset with Trendelenburg position	0	–
Additional features		
Height adjustment	✓	–
2 adapter sleeves at the head end for patient lifting pole/infusion stand	✓	–
Emergency backrest lowering mechanism	0	–
Bed extension approx. 20 cm	0	–
Linen holder ¹⁾	0	–

✓ = Standard feature
0 = Available as an option
– = Not available

¹⁾ not possible with integrated combined safety sides



Forensic medicine

Requirements for security-sensitive areas

Caring for people in prison does not differ fundamentally from caring for people outside penal institutions. People in prison also need medical care.

There are special features and challenges, however, when it comes to the daily work routine.

The choice of suitable hospital beds is influenced by the fact that safety and security are paramount.

The three variants of the Deka psychiatric care bed for prisons described below are based on the experience of our customers and on practical requirements in closed psychiatric wards.

In the interest of ensuring the highest possible level of safety, none of the versions have motorised adjustments.



Deka

Variant 1

- Posey belt fixing points on the mattress base frame and additionally on the adjustable backrest
- Hydraulic height adjustment
- Headboard and footboard available in two design variants and permanently screw-fixed in place
- Four-section mattress base and option of locking the leg rest adjustment
- Option of replacing the castors with feet to fix to the floor



Deka

Variant 2:

- Stainless steel standard sliding rails on the mattress base frame with subdivisions for fixing posey belts, and additional securing brackets on the backrest
- Headboards and footboards in the Classic design only; can be folded down and secured with magnets
- Hydraulic height adjustment
- Four-section mattress base and option of locking the leg rest adjustment
- Option of replacing the castors with feet to fix to the floor



Deka

Variant 3:

- Two-section mattress base
- Chassis with four feet
- Not height adjustable; not mobile
- Securing brackets on the mattress base frame and on the backrest, as in variant 1



- ◀ Full-length safety sides, as a hook-on version for retrofitting, or permanently fixed with screws.



Deka

The stable bed for challenging everyday conditions

The safety of patients and nursing staff is paramount in the case of all variants of Deka psychiatric beds. In addition to waiving an electric drive, mechanical adjustments can also be limited, if desired, or access can be made more difficult to the operating levers of the gas springs and hydraulic system. The mattress base is made of welded-in steel wire mesh panels that are secured against removal.

Two headboard/footboard designs to choose from: The Classic headboard/footboard has a bar that runs around the board and provides plenty of space for attaching accessories. The same applies to our Valero headboard/footboard, which has the added benefit of an attractive, modern design. Both models can be secured against inadvertent removal.



▲ Classic headboard/footboard.



▲ Classic footboard, with reinforcement.



▲ Classic footboard, folds down through 180°.



▲ Valero headboard and footboard.



▲ Feet for fixing the bed to the floor.



▲ Brake lever permanently secured with screws.



▲ Securing brackets on mattress base frame and backrest.



▲ Standard sliding rail on mattress base frame.



▲ Closed connection sleeves.



▲ Height pedal secured with Segufix system.



▲ Height pedal secured with locking pin.



▲ Alternative actuation of the rests on the gas springs.

Performance characteristics of restraint beds

	Deka Variant 1	Deka Variant 2	Deka Variant 3
Mattress base frame	With securing brackets	With standard sliding rail	With securing brackets
Mattress base	Wire mesh base, fully welded in, 4 sections	Wire mesh base, fully welded in, 4 sections	Wire mesh base, fully welded in, 2 sections
Backrest with welded on brackets for securing posey belts	✓	✓	✓
Lockable thigh rest and lower leg rest	0	0	-
Headboard and footboard			
Classic, fixed	0	-	0
Classic, foot end, fixed and reinforced	0	0	0
Classic at foot end, fixed and reinforced, folds down through 180°	0	0	0
Classic, folds down through 180°	0	✓	-
Valero, fixed, stainless steel posts	0	-	0
Valero, removable, with securing element for full-length safety side	0	-	0
Safety sides			
without	✓	✓	✓
Hook-on full-length safety side	0	0	0
¾ safety side	0	-	0
Internal adapter sleeves for patient lifting poles			
Oval or round pole	✓	✓	✓
Closed connection sleeves	0	0	0
Additional features			
Height adjustment	✓	✓	-
Mattress base tilt	0	0	-
For fixing to floor	✓	✓	✓
With individually locking castors	-	-	0
With castors and central castor locking	0	0	-
Pump pedal secured, removable without using tools	0	0	-
Bed extension	0	0	0
Linen holder	0	0	0

Useful extras

This may also interest you!



- ▲ The **slimline Quado bedside cabinet** is ideally suited to everyday hospital use in rooms with limited space. Whenever the overbed table is not in use, it can be folded away easily and softly. The Quado can be transported with the bed by simply hanging it onto the headboard or footboard.



- ▲ The **new Aparto bedside cabinet range** combines modern care trends with a homelike ambience and tailored design options. If an Aparto model is fitted with an integrated overbed table, its body can be rotated in just seconds. In this case, the bedside cabinet can be used on either side of the bed and it adapts to changing needs and circumstances in the room.



Are you familiar with our furniture range, including our tables and chairs? Please feel free to get in touch with us!

STIEGELMEYERGROUP



Solutions for people who care

The Stieglmeyer Group is Germany's leading manufacturer of medical beds for hospitals, nursing homes and homecare. With modern bedside cabinets, furniture and digital assistance systems, it is also an innovative provider of comprehensive solutions. The family-run company successfully develops offers for international markets in more than 100 countries.

Our strengths:

- Product development at our Herford headquarters in Germany – production in four modern factories in Germany and Poland
- Factory in China for the Chinese market
- Subsidiaries and sales partners in many countries
- Top-class service with reliable spare parts supply
- Training and process optimisation for our customers
- Sustainable action with strong social commitment

Our expertise

- Hospital
- Nursing home
- Homecare
- Lifestyle beds
- Furniture systems
- Service

Our values

- Experienced
- Human
- Reliable
- Solution-focused
- Open-minded
- Confident

1 000
EMPLOYEES
WORLDWIDE

11
LOCATIONS
in Germany, Poland,
the Netherlands, Belgium,
France, Finland,
South Africa and China

Tailor-made
OFFERS
for all
MARKETS

MORE THAN
100 000
beds produced
every year

UP TO
15-YEAR
spare part availability

DISTRIBUTION
in more than
100 countries

INTERNATIONAL
CERTIFI-
CATION

GERMAN
ENGINEERING

Our brands





STIEGELMEYERGROUP

Solutions for people who care



Hauptsitz / Headquarters

Deutschland / Germany
Stieglmeyer GmbH & Co. KG
Ackerstraße 42
32051 Herford
Phone +49 (0) 5221 185 - 0
Fax +49 (0) 5221 185 - 100
info@stieglmeyer.com
www.stieglmeyer.com



Hauptsitz / Headquarters

Deutschland / Germany
Burmeier GmbH & Co. KG
Industriestraße 53
32120 Hiddenhausen
Phone +49 (0) 5223 9769 - 0
Fax +49 (0) 5223 9769 - 090
info@burmeier.com
www.burmeier.com

France

Stieglmeyer SAS
16 Avenue de l'Europe
Bâtiment SXB1
67300 Schiltigheim
Phone +33 (0) 4 7238 - 5336
contact@stieglmeyer.fr
www.stieglmeyer.fr

South Africa

Stieglmeyer Africa (PTY) Ltd.
The Estuaries - Century City
Cape Town, 7441
Phone +27 (0) 21 555 - 1415
Fax +27 (0) 21 555 - 0639
info.southafrica@stieglmeyer.com
www.stieglmeyer.co.za

Nederland / Netherlands

Stieglmeyer BV
Peppelenbos 7
6662 WB Elst
Phone +31 (0) 345 581 - 881
info@stieglmeyer.nl
www.stieglmeyer.nl

Suomi / Finland

Stieglmeyer Oy
Ahjokatu 4 A
15800 Lahti
Phone +358 (0) 3 7536 - 320
stieglmeyer@stieglmeyer.fi
www.stieglmeyer.fi

Polska / Poland

Stieglmeyer Sp. z o.o.
86-212 Stolno
Grubno 63
Phone +48 56 6771 - 400
Fax +48 56 6771 - 410
info.stolno@stieglmeyer.pl
www.stieglmeyer.pl

België / Belgique / Belgium

Distrac Group
3320 Hoegaarden
Bleyveld 14
Phone +32 (0) 16 76 82 82
info@distracgroup.com
www.distracgroup.com

China

Stieglmeyer China Co. Ltd.
Poly Center, No. 8 South Wenhua Road
Dongping New Town, Shunde District
528300 Foshan
Phone +86 757 29878710 - 0
info@stieglmeyer.cn
www.stieglmeyer.cn



stieglmeyer-group.com